



EMODIAL®



EXIT-POCKET® WP



EMODIAL®

Via Finati, 4-4/A - Ferrara - ITALY
Tel. +39 0532 978198 - Fax +39 0532 978414
www.emodial.com - info@emodial.com

**Waterproof Pocket
for Catheter Protection**

Ref. GB0013 - Rev. 10/2023

EXIT-POCKET® WP

Waterproof Pocket for Catheter Protection

DESCRIPTION

EXIT-POCKET WP is a waterproof pocket closed on four sides provided with adhesive hole to be fixed to the skin.

INDICATIONS

Containment of catheter lumens during patient's hygiene procedures (shower and bath) and during water activities (diving, sea, swimming pool).

PURPOSE


Designed to protect the exit-site from water contact. The device is completely waterproof: if well-positioned it protects the exit-site from contact with water preventing the possibility of microbial contamination.

BENEFITS

- Quick and easy placement.
- Comfortable fit.
- Helps to improve the quality of life of catheterized patients.
- Easy and fast removal.

DIRECTIONS FOR USE


1. Insert catheter lumens or the protective device that contains them into the pocket hole.
2. Remove the protective sheet from the adhesive.
3. Place the lower margin of the adhesive hole in correspondence to the lower part of the exit-site, or of the dressing if present, and complete the fixing from bottom to top, avoiding wrinkles or airlocks.
4. Wait 10 minutes from device application.
5. Take shower or bath paying attention not to rub the area near the adhesive.
6. To remove the device, gently tear out a small flap of adhesive from the skin and carefully pull upwards with gentle movements in small strokes, taking care not to pull the catheter cannulas.

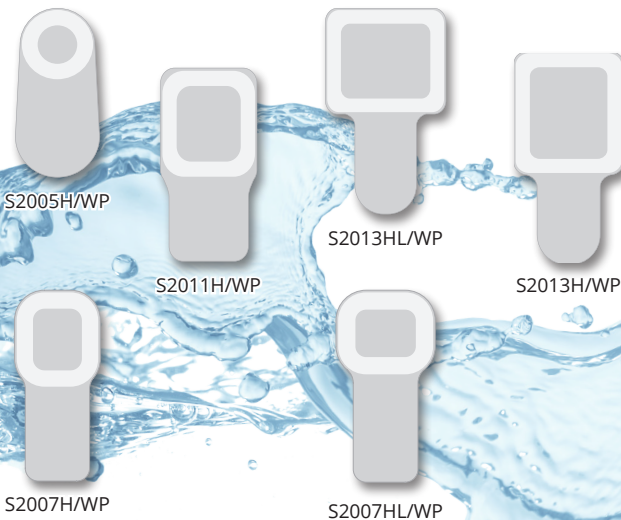

REFERENCES

CODE	SIZE (cm)			INDICATION	PCS / BOX
	Hole	L	H		
S2005H/WP	ø 5	11	22	Bath and Shower	50
S2011H/WP	10 x 9	13	26	Bath and Shower	50
S2013HL/WP	13 x 11	17,6	30	Bath and Shower	50
S2013H/WP	11 x 13	15,6	30	Bath and Shower	50
S2007H/WP	5,5 x 7	10	22	Bath and Shower	25
S2007HL/WP	7 x 5,5	12	22	Bath and Shower	25

CLASSIFICATION • CE MEDICAL DEVICE
STERILIZATION • NOT STERILE

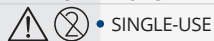

• SINGLE-USE